

Point Thumb - OI by Point Designs

Guide for Users


These instructions are for users of the *Point Thumb - OI*. For other questions, contact your prosthetist. For questions about the surgical attachment (abutment), contact the implant manufacturer.

Use of the Point Thumb - OI


The *Point Thumb - OI* is designed for people with a bone-anchored implant who are medically cleared for a prosthesis. It is designed for light everyday activities, like simple tasks at home.

The *Point Thumb - OI* works by pushing the back of the thumb forward, allowing it to bend and lock into one of five positions. The position of the thumb can be adjusted using the other hand, fingers on the same hand, or surfaces around you. You can adjust or reset the position as follows:

1. Bend the thumb fully forward, then let it return to a straight position to reset it.
2. Press the button on the back of the thumb to straighten it gradually

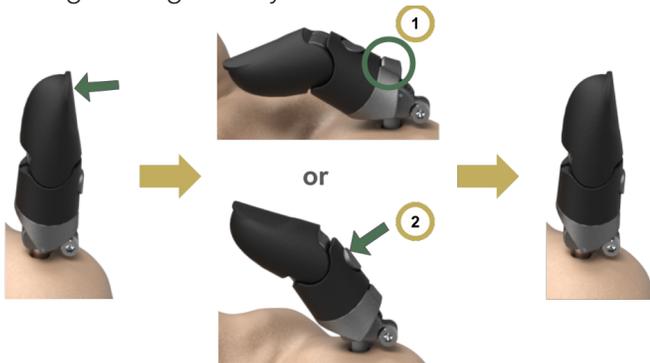


Once installed, the *Point Thumb - OI* can be rotated relative to the abutment to increase the number of achievable grasps. To rotate the thumb, loosen the clamping screw with the provided 2.5mm hex driver (1), grab the thumb and rotate it in the desired direction (2), and then tighten the clamping screw (3).

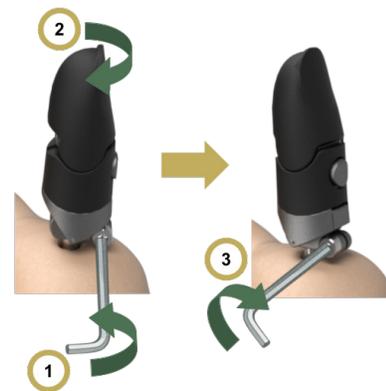

Putting on & taking off the Point Thumb - OI

To attach the *Point Thumb - OI*, loosen the clamping screw (1) and place the thumb onto the abutment (2). Rotate it to a position that feels best for picking up everyday items (3). Use the 2.5 mm hex driver provided to tighten the screw in the direction shown until snug (4). The thumb should not wobble on the abutment and should not pull off of the abutment when gently tugged.

To remove the *Point Thumb - OI*, loosen the screw and pull the thumb off of the abutment. Note that the screw does NOT need to be removed completely and should stay inserted in the *Point Thumb - OI*.

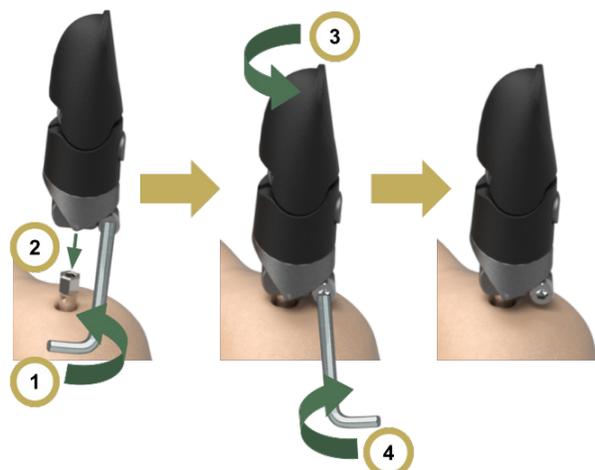

Tighten gently up to a torque of 0.25Nm. Do not overtighten.

Cleaning

The *Point Thumb - OI* can be cleaned using the methods below. Remove the prosthesis before cleaning.

- To remove dust or small particles, gently blow forced air over the area.
- Rinse the prosthesis with fresh water, especially after exposure to salt water or other harsh environments.
- If needed, clean with mild hand soap that is free of harsh detergents, strong fragrances, or high-pH ingredients.
- Dry thoroughly after washing.

For cleaning the abutment and surgery site, follow the instructions from your surgical care team and the implant manufacturer.

Maintenance and Replacement Parts

The *Point Thumb - OI* does not need regular maintenance. If it is damaged or stops working, contact your prosthetist to request a repair from Point Designs.

The *Point Thumb - OI* includes one extra screw to secure the adapter piece.



Warnings



WARNING: The *Point Thumb - OI* can safely support 80N (18lbf), but the total limit also depends on the abutment and surgical implant. Contact the implant manufacturer for details.



WARNING: Remove the *Point Thumb - OI* before heavy or strenuous activities, such as lifting, pushing, or pulling with force.



WARNING: If any part of the *Point Thumb - OI* is damaged or not working, stop using it and contact your prosthetist.



WARNING: The *Point Thumb - OI* has metal parts that conduct heat and electricity. Keep it away from hot surfaces, exposed wires, and strong magnets to avoid damage or injury.



WARNING: The *Point Thumb - OI* is not meant for continuous wet use. After water exposure, dry it thoroughly by air drying, patting with a towel, or gently blowing forced air.



WARNING: The *Point Thumb - OI* has moving parts that can pinch. Keep loose skin and clothing away from the moving parts.